Safety Communication

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Safety Notice of Portable Ventilator.

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Problem:

When Astral ventilators are used with the internal battery as the sole power source, it is important to note that any performance issue with the internal battery could potentially lead to cessation of therapy.

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Remedy Action:

ResMed is updating the User and Clinical Guides to ensure the safe use of ventilators when not on AC power. The labelling for the ventilators is being modified with some warnings.

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Recommendation:

- The internal battery is NOT intended to serve as a primary power source. It should only be used when other sources are not available or briefly when necessary.
- Astral ventilators must be returned to a service center for routine two-year maintenance which includes a scheduled battery replacement.
- Share this information with Healthcare Professionals in your facility.

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All Astral 100, Astral 100SC and Astral 150 ventilators

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You **should** be aware of the mentioned risks in the notice and **contact** the Authorized Representative for corrective action.

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Healthcare Professionals should **report** any adverse events suspected to be associated with affected devices above (or other Medical Devices) to:

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National Center for Medical Devices Reporting ([NCMDR Website](#)).

Medical Devices Sector
Saudi Food and Drug Authority
Postal Address: Saudi Arabia - Saudi Food and Drug Authority (3292)
North Ring Road - Al Nafal Unit (1)
Riyadh 13312 - 6288
Tel: +966 (11) 2038222 Ext: 2406

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Fax: +966 (11) 2757245

For latest published Recalls/Alerts, please visit
NCMDR website:

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